






MEDI DERMA-S

Skin Protection Guide

Use this table to help you identify the skin damage type and to choose the most effective product from the **MEDI DERMA-S** skin protectant range.

1. SKIN ASSESSMENT

Type of Skin Damage	Barrier Cream <i>(Skin Protection + Moisturisation)</i>	Barrier Film <i>(Skin Protection)</i>
 Intact skin at risk of breakdown due to fragility (eg: elderly skin, skin stripping due to frequent dressing changes, stoma site)	As required	At each dressing/ application change
 Intact, irritated skin at risk of skin breakdown due to incontinence (urine and/or faecal)	As required After each episode of incontinence	Every 72 hours
 Irritated broken skin due to incontinence (urine and/or faecal)	As required After each episode of incontinence	Every 48 to 72 hours
 Macerated periwound skin due to excess exudate	N/A	At each dressing change
 Moisture-related skin damage (skin folds) due to perspiration <small>Note: Seek further medical advice if you suspect intertrigo, which might require topical steroid and, possibly, anti-fungal or antibiotic treatment.</small>	N/A	Up to 3 times a day

Reproduced with permission from Wounds UK. Skin Protection Quick Guide © Wounds UK 2013.

2. PRODUCT CHOICE

FOR SKIN PROTECTION + MOISTURISATION

USE CREAM

MEDI DERMA-S
Barrier Cream

Healthcare Grade

FOR SKIN PROTECTION

USE FILM

MEDI DERMA-S
Medical Barrier Film

Silicone Based, Healthcare Grade

CREAM AND FILM FORMATS
BOTH SUITABLE FOR USE ON INTACT OR DAMAGED SKIN

Format	Product Code	PIP Code
2g Sachets (20 Pack)	60338	341-3317
28g Tube	60628	388-3121

Format	Product Code	PIP Code
30ml Pump Spray	60796	389-7121
1ml Applicators (5 Pack)	61076	362-8716
3ml Applicators (5 Pack)	61090	362-8724